

Quality time for better care

"Quality time for better care" is the brand promise of Terumo Medical Care Solutions.



HOSPITAL CARE

Terufusion™

Infusion System



Software Packages

Software Package Basic

Drug Library Manager + Data Communicator

Golden standard for daily operations



Create and edit drug libraries

- More than 3000 drugs available
- Up to 30 customizable profiles for different wards and needs
- Set and restrict critical parameters





Drug name with Tall Man letters	To distinguish from
DOBUtamine	DOPamine
buPROPion	busPIRone
chlorproMAZINE	chlorproPADIME
CEFOtaxim	CeftAZIdim
ATEnolol	AceBUTolol
cycloSPORINE	CYCLOserine
DimenhyDRINATE	DIPHENHYDRamine

ISO 26825 – Drug coding

- Label vs display safety crosscheck
- Many colours and patterns available

“Tall Man” formatting

- Recommendation for look alike drug names

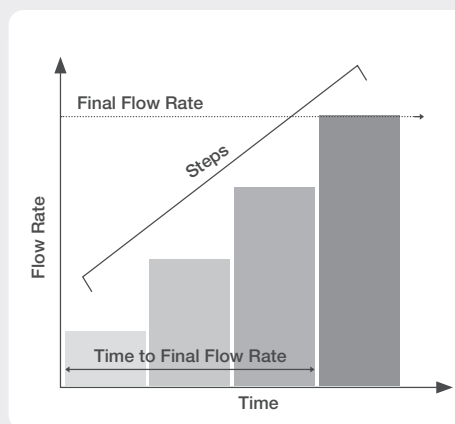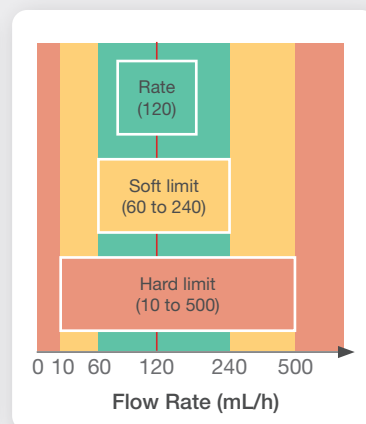

Clinical Advisory

- Customized clinical advisories with colour code

Soft and Hard Limits

- Prevention of dose errors

Advanced Infusion Protocols

- Multi-step, Intermittent, Delayed Start or Combined with another pump




Software Package Professional

Drug Library Manager + Data Communicator + Dose Analyzer

Software tools for continuous improvement



Loop of excellence

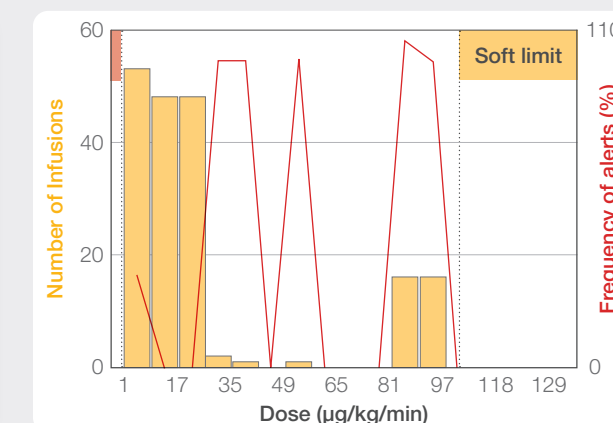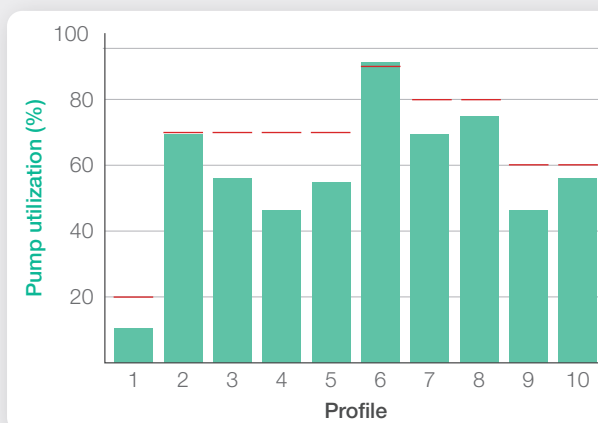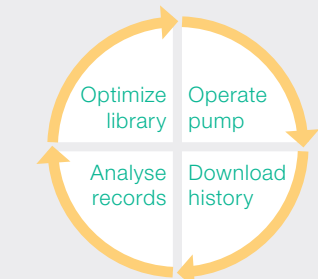


Report of pump use

- The right number of devices in each ward
- Fix & check your targets


Drug Library Analysis

- The right infusion protocol at bedside
- Check & adapt infusion parameters

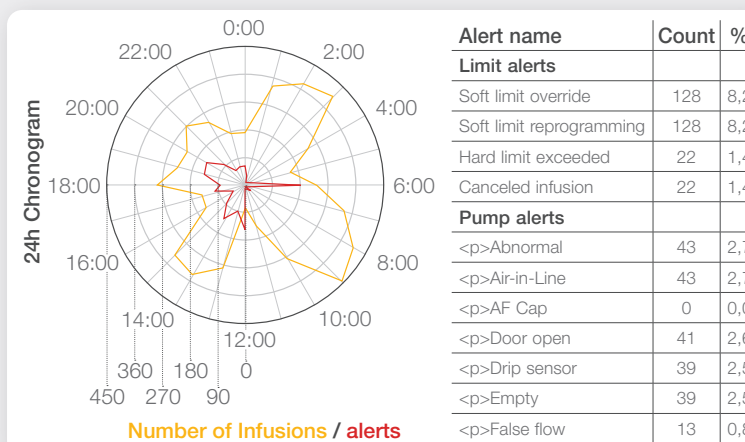

Alert Statistics

- The efficient setup of alarms in each room

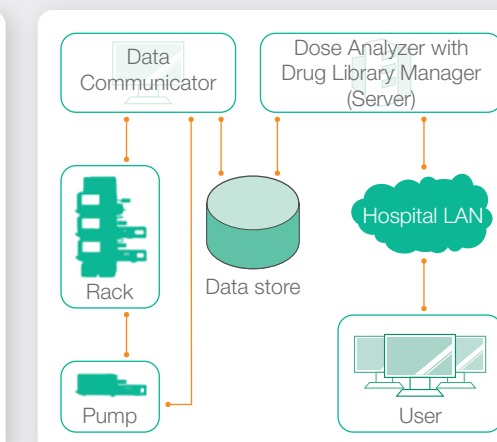

Connectivity

- A browser can be used to access the software and data to review the analysis results.

System requirements

Server	
Operating System	Microsoft® Windows® 7 SP1 Professional 32-bit (x86) / 64-bit (x64) Microsoft Windows 7 SP1 Enterprise 32-bit (x86) / 64-bit (x64) Microsoft Windows 7 SP1 Ultimate 32-bit (x86) / 64-bit (x64) Microsoft Windows 8.1 Professional 32-bit (x86) / 64-bit (x64) Microsoft Windows 8.1 Enterprise 32-bit (x86) / 64-bit (x64) Microsoft Windows 10 32-bit (x86) / 64-bit (x64) Microsoft Windows Server® 2008 SP2 32-bit (x86) / 64-bit (x64) Microsoft Windows Server 2008 R2 SP1 64-bit (x64) Microsoft Windows Server 2012 64-bit (x64) Microsoft Windows Server 2012 R2 64-bit (x64)
Memory	2 GB or more recommended
CPU Clock	Intel® 2 GHz or more recommended
HDD	80 GB or more free space recommended
Device	DVD-ROM drive
Web Server	Internet Information Services 7.0/7.5/8.0/8.5/10.0

User	
Memory	512 MB or more recommended
CPU Clock	Intel 1 GHz or more recommended
Screen resolution	1024 x 768 pixels or more
Operation conditions	Temperature: 10 to 40°C Relative humidity: 20 to 90% RH (non-condensing)
Storage and transport conditions	Temperature: -20 to 60°C Relative humidity: 10 to 95% RH (non-condensing)

Compatible equipment

Type	Model	Communication possibilities
Infusion Pumps	TE-LM830 / TE-LM835	Power Status, High and Low Priority Alarms, Operation Monitor (WLAN)
Syringe Pumps	TE-SS830 / TE-SS835	Power Status, High and Low Priority Alarms, Operation Monitor (WLAN)
Communication Rack (Extension)	TE*RS800N03 (TE*RS811N)	Power Status, High and Low Priority Alarms, Operation Monitor (WLAN, LAN, RS-232C)

Ordering information

Software	Contents	Product Code
Software Package Basic	Drug Library Manager, Data Communicator	TE*SW800BE
Software Package Professional	Drug Library Manager, Data Communicator, Dose Analyzer	TE*SW800PE

Full focus infusion system

Terumo offers a comprehensive portfolio of solutions for the complete infusion line-up. User-friendly syringe and volumetric pumps and racks, as well as a wide range of highly accurate, easy-to-use disposables, such as an extended range of infusion sets and syringes.

